AlfaTM

TRIPLE TRAY[®]

Alginate Full-Arch Closed-Mouth
Impression Tray

TO SIMULTANEOUSLY OBTAIN
THE MAXILLARY IMPRESSION,
MANDIBULAR IMPRESSION
AND BITE REGISTRATION.

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INDICATIONS: All alginate impressions - ideal for fabrication of custom trays; tooth whitening trays, temporaries, study models, mouth guards/night guards, removable prosthetics and orthodontics.

DIRECTIONS FOR USE

- 1) Select appropriate size tray. Medium fits most adult patients. Premier logo on top of handle ensures proper maxillary/mandibular orientation.
- 2) Try in selected tray by inserting into the patient's mouth. It should fit passively, with the crossbars distal to the most posterior teeth in arch. As with any impression making technique, ensure patient can breath comfortably through nose with mouth closed.
- 3) Prepare 2-3 scoops of alginate, depending on tray size. Load alginate into tray along outer walls to ensure it flows into retentive undercuts. Remove excess; alginate should extend just beyond tray wall.
- 4) Place loaded tray into the mouth. Retract the cheek with a dental mirror to provide easier access. The tray should be centrally placed and not impinge on tissue. Have the patient place their tongue into the posterior notch of the tray.
- 5) If necessary, assist the patient by pulling their lips beyond and over tray wall as they close into a natural bite.
- 6) Remove the set alginate impression. Place your finger on either purchase point. Press down and out as you ask the patient to open. Once the suction is broken, the tray will be easily removed. Disinfect and inspect the impression. Pour the stone as per the manufacturer's directions.

Alfa Triple Tray can also be used with alginate substitute vinyl polysiloxane (VPS) material. Do not use a rigid setting or heavy body VPS material. Block out any undercuts and do not use if the patient has loose restorations of any kind.

LAB TECHNIQUES

Pour up the impressions as per the stone manufacturer's directions.

- 1) Mix the stone and pour up one arch, making sure that no bubbles are incorporated into the stone.
- 2) Cover the opposite arch with a damp towel and pour up as soon as the first arch is set. To save time consider using a fast-set stone.

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