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
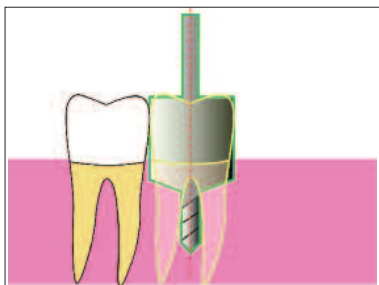
MD Guide - Instructions for Use

MD Guide is designed to create a surgical pilot hole to place an implant accurately. When properly placed against the adjacent tooth, MD Guide provides precise mesio-distal guidance and allows for alignment of the implant with the future prosthetic tooth. This prevents any risk of misplacing the implant.


For Single Implant placement

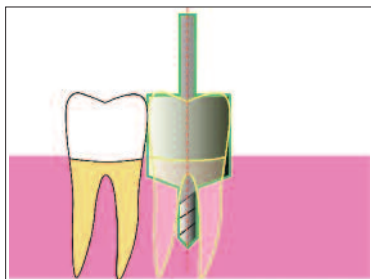1. Estimate the mesio-distal dimension of the missing tooth. The 6 mm size is recommended for an incisor, a 9 mm size for a premolar and a 10.5mm or 12 mm for a molar.
2. Place the MD Guide matching the proper dimension into your surgical handpiece.
3. Simultaneously place the MD Guide cylinder against the adjacent tooth and drill tip against the bone crest (or against the gingiva in the case of flapless surgery).
4. Align the MD Guide with the adjacent tooth axis to make them parallel.
5. Start the motor at approximately 1200 rpm.
6. Drill until the cylinder hits the gum or the bone crest.
7. Remove the MD Guide.

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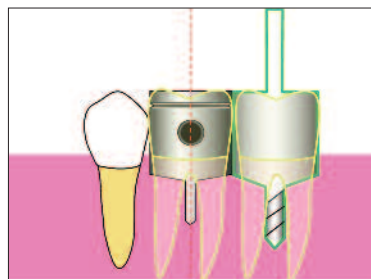1. Follow steps 1 to 7 to create the pilot hole for the first implant.
2. Introduce a non-drilling guide (analogue) matching the mesio-distal dimension of the missing tooth into the first pilot hole. This non drilling guide will simulate the presence of the future crown.
3. Estimate the mesio-distal dimension of the next missing tooth.
4. Place the MD Guide matching this dimension into your handpiece.
5. Place simultaneously the guide against the adjacent non drilling guide and the drilling tip against the bone crest (or against the gingiva in case of flapless surgery).
6. Align the MD Guide with the adjacent non drilling guide axis to make them parallel.
7. Start the motor at approximately 1200 rpm.
8. Drill until the cylinder hits the gum or the bone crest.
9. Remove the MD Guide.
10. Repeat steps 2 to 9 for more implants.

Please note: when storing the drills into the original surgical box, place the drilling tip up.

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