protrain

endodontic system for training

Patented by Dr. Riccardo Tonini

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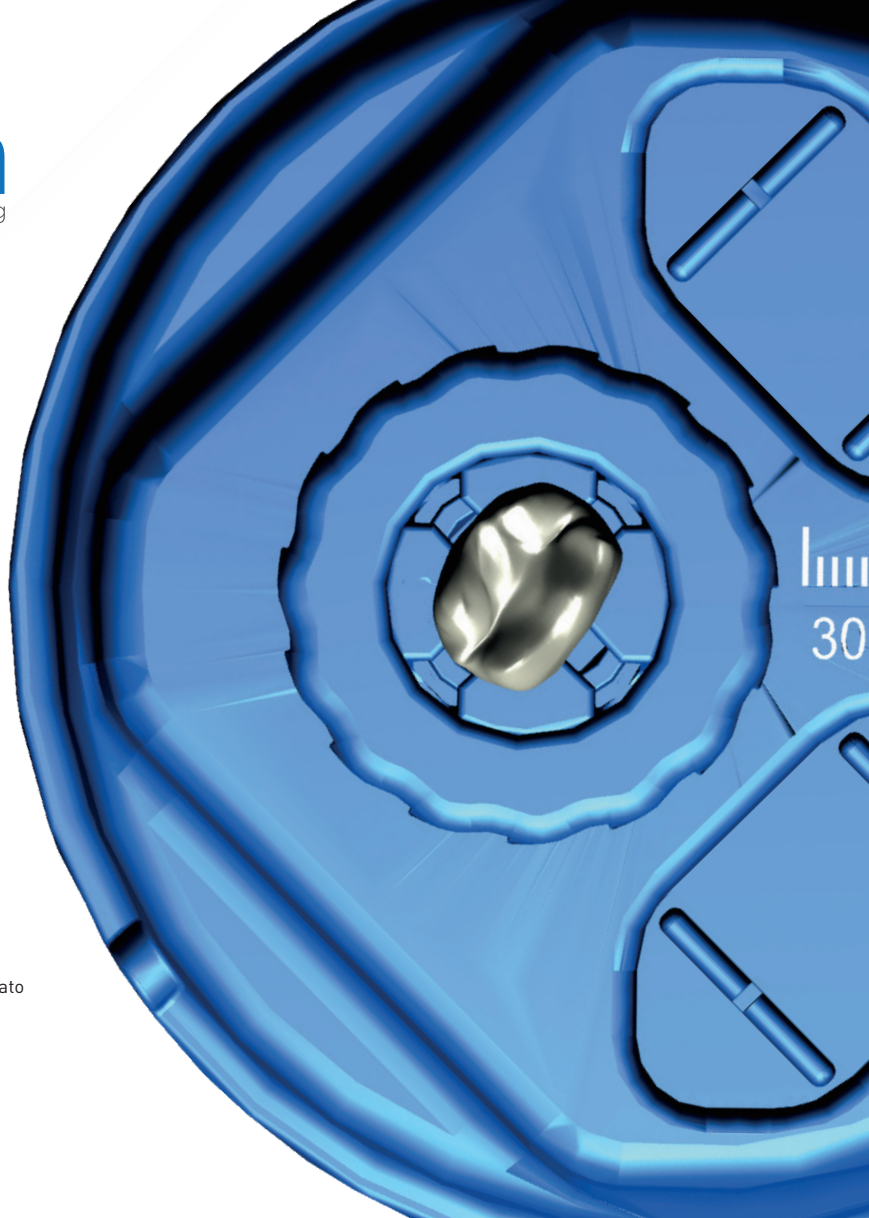
Producer
Simit Dental Srl
www.simitdental.it

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
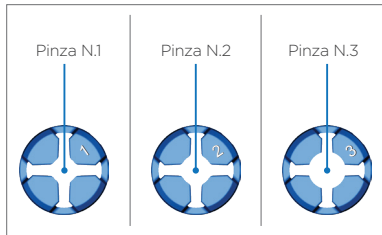
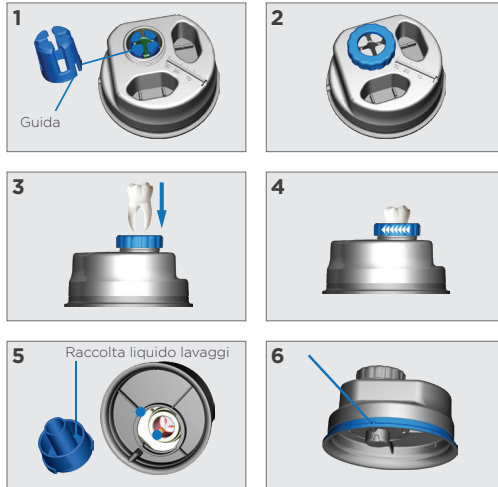
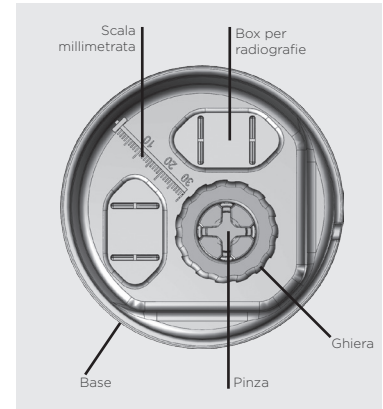
ENDO-PAD ORGANIZER

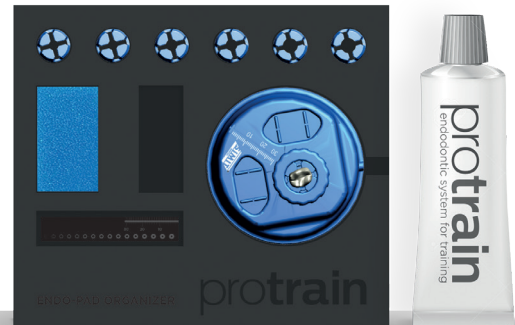ProTrain is the only device designed and patented specifically for the endodontic training that allows the operator to perform endodontic therapy on extracted teeth as if they were still alive. The extracted tooth is locked by a nut-ring to allow the operator to perform all the necessary operations for a correct endodontic treatment.

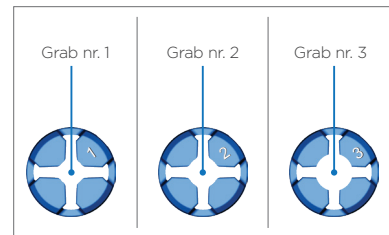
With ProTrain it's possible to connect the apex locator, to do digital X-Ray radiographies with two projections of 90°, to measure the files with millimetric scale that are used and to do perform root canal washes.

The system allows locking the most of the extracted teeth

It's suggested to use the saline solution for the endocanal washing.
If there are any problems in measuring the root canal, please be sure that the root is dipped in the wet sponge.


Three are 3 different size of Pliers:

- Plier **nr.1** for small diameter teeth (incisive, canines, premolars)
- Plier **nr.2** for medium diameter teeth (premolars, canines)
- Plier **nr.3** for large diameter teeth (molars)

Instruction for use

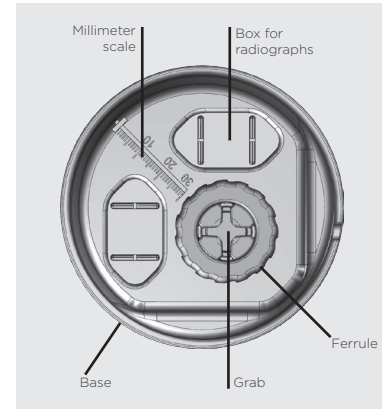
1. Once selected the tooth, insert the correct grab.
2. Screw the nut-ring until the grab starts to lock.
3. Insert the tooth into the grab leaving out only the crown.
4. Screw the nut-ring and lock the tooth.
5. Take the sponge, present in the package, and wet it with physiological solution.
6. Insert the wet sponge into the tank and lock it in the Protrain, being sure that the two spots dovetail and then turn the tank counter-clockwise.
7. Insert the metal bar through the guide until it is inserted in the tank connect the bar to one of the apex locator connectors.

Use intraoral radiographic film 30 x 40; place the film upright in the Protrain boxes (exposition: 0.20 kV). It is possible to perform digital radiography with sensor (exposition: 0.08 kV)

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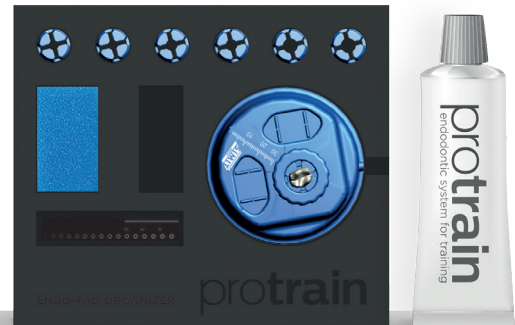
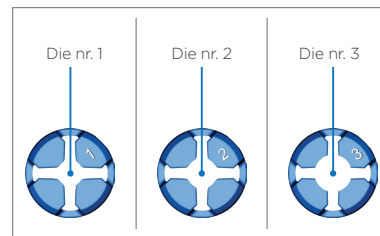
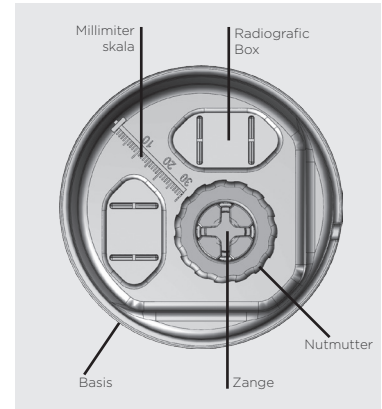
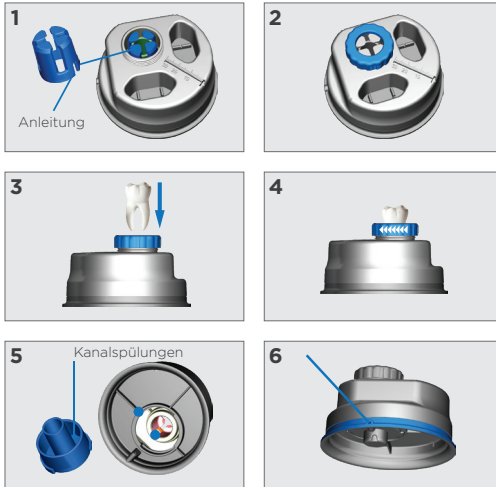
ENDO-PAD ORGANIZER

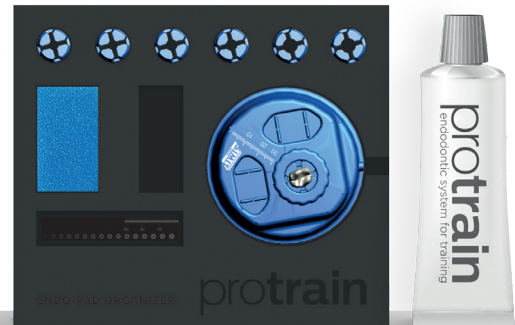protrain
endodontic system 4 training

www.protrain.it

IVAN ILIC srl

Via L. Muratori, 46/11 - 20135 Milano (Italy)

TEL: +39 02 36 512 990 - 02 55 016 500

ilic@ilicdental.com

www.ilicdental.com