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SHOFU All Ceramic Preparation Kit

**For Preparation of
All Ceramic Restorations**

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www.ilicdental.com

SHOFU All Ceramic Preparation Kit

The demand for ceramics in dentistry is at an all time high. Newer ceramic materials available today produce superb ceramic restorations. This is challenging the dental professional to consider newer materials, esthetics, function, and clinical expertise when preparing an all ceramic restoration.

Under the guidance of Dr. Takao Maruyama of Osaka University, Dr. Yoshihiko Mutoke and Shofu have developed an all ceramic preparation system utilizing diamond burs to prepare an all ceramic restoration.

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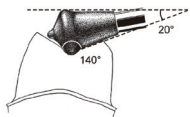
Note: V = Very Fine

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0899-1, 899V-1 Tapered 20° angle (at inverted conical shape). Use to reduce an upper premolar and the occlusal area of an upper and lower molar to maintain the occlusal space. Keep the upper axis of the diamond bur parallel to the occlusal surface to prepare a 140° angle, the average occlusal cusp angle of a molar. By altering the inclination of the diamond bur, adjust the cusp top angle to accommodate the opposing tooth, the upheaval shape of the diamond bur produces a concave prep making the necessary space for the epōnt of a eroven lta lœ mod b h fiñh of b ovrrall dg of b occlusal and axial angle of the molar.

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8088V-1 Use to eliminate the free enamel in the cervical area. User a lower speed to avoid chipping the enamel.

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
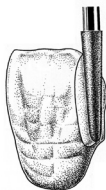
1. Slice cut the interproximal area of the anterior tooth (Fig. 1-1). Use the long axis of the bur (Fig. 1-2). After slice cut, retract tissue with retraction cord.

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1-1

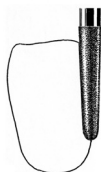1-2

2. Reduction of incisal edge (Fig. 2-1). Reduce the incisal edge 1.5 – 2.0 mm. Angle the diamond toward the lingual to stay parallel with the incisal line.

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
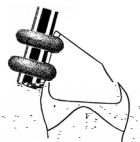
3-1

3-2

8. Prepare the shoulder margin. Smooth and finish the chamfered cervical area (Fig. 8)

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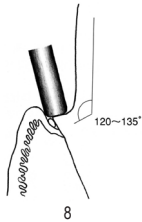
8

9. Smooth and finish the shoulder preparation using the outside 1/3 of the bur cutting edge that is away from the long axis of the prep. Move the bur in the direction of the rotation (Fig. 9).

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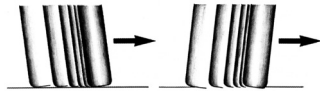9

10. Place the bur vertically on the shoulder margin surface, not parallel to the long axis of the tooth preparation (Fig. 10).

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
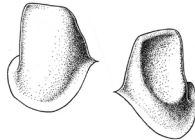
10

11. Completed preparation (Fig. 11).
Check the final preparation (Fig. 12 and 13).

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
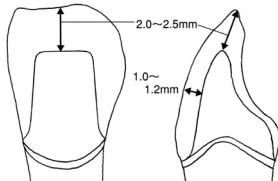
1. Preparation of lower premolar (1st bicuspid).
Interproximal view (Fig. 14).
Buccal view (Fig. 15).

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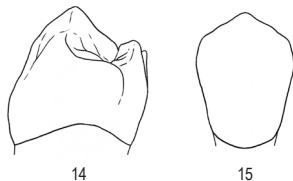2. Reduce the four surfaces of the premolar. A slight undercut will occur while following the lingual inclination of the premolar. The bur, when placed correctly against the occlusal plane, will tip toward the buccal surface (Fig. 16).

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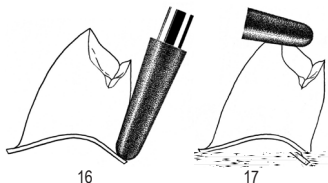
3. Reduce the apex of the buccal cusp following the inclined plane angles (Fig. 17).

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
4. Utilizing the premolar anatomy. The occlusal surface is prepared similar to an anterior preparation (Fig. 18).

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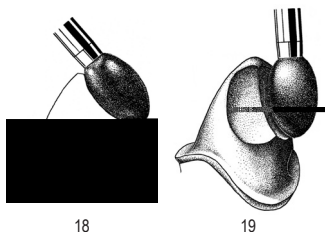5. To make broad mesial and distal pits in the occlusal surface, reduce toward each interproximal direction (Fig. 19).

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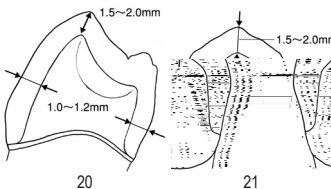6. Completed preparation. Interproximal view (Fig. 20).

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7. Completed preparation. Buccal view (Fig. 21).

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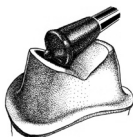
22

23

2. Reduce occlusal surface to create occlusal space (Fig. 24-1). Use shape of the bur top to cut the central groove and to shape the side areas for the pits of the crown (Fig. 24-2).

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
24-1

24-2

3. Use the nipple part of the bur top to smooth and finish the overall angle of the occlusal axial angle (Fig. 25-1 and 25-2).

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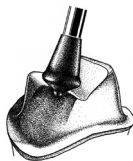
25-1


25-2

4. Completed preparation. Interproximal view (Fig. 26).

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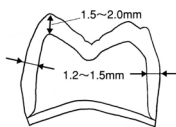26

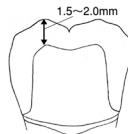27

5. Completed preparation. Buccal view (Fig. 27).

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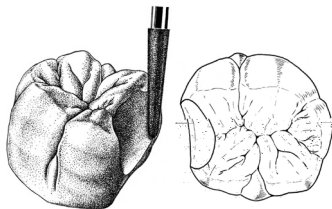
1. Cut the step for the box preparation in the interproximal area (Fig. 28).
To make a clear margin on the buccal and lingual surface, cut with the bur counter-clockwise toward the apex of the tooth. Move from the buccal toward the lingual (check the occlusal contact to determine the form of the preparation prior to cutting the prep).

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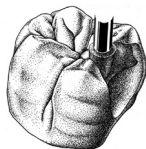
29

30

2. Form the box preparation on the occlusal surface (Fig. 31).
Prepare a gently curved preparation, not a dove-tail preparation (Fig. 32).

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31

32

ess e at l a t 1.5 mm from th fo a to th floor of th preparation (Fig. 33).
 The box depth of the prep is cut 2.5 - 3.0 mm considering the length of the bur (3.5 mm) (Fig. 34).

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33

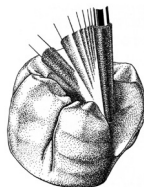
34


3. Round the corner of pulpal and axial walls. (Fig. 35-1 and 35-2).

Rounding of the box prep will strengthen and produce an excellent inlay (Fig. 36).

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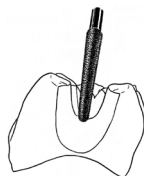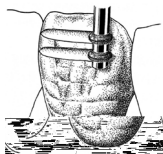
36

Anterior Tooth/Frontzahn

1. Retract tissue prior to preparation with retraction cord.

Cut a lateral guide groove 0.5 mm in depth, then outline the groove with a marking pen (Fig. 37).

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
37

2. Following the drawn outline, prep the surface (Fig. 38).

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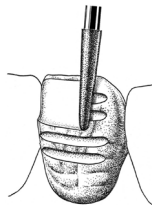
38

3. In order to maintain proper thickness of the veneer, keep the bur vertical to the cervical tissue (Fig. 39).

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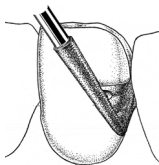
39

4. Prepare the embrasure at the lower interproximal area (Fig. 40).

Prep the interproximals to the contact point. Prep from the facial in order not to expose the margin on the mesial or distal sides (Fig. 41). Prep at the correct angle to avoid undercuts mesially and distally.

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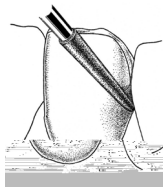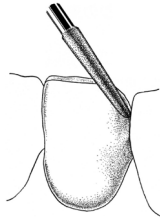
41

5. Prepare the embrasure in the upper interproximal area (Fig. 42).

Prep the upper interproximal areas the same as the lower interproximals to avoid enamel tipping, move the bur in the direction of rotation.

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42

6. Prep the incisal margin (Fig. 43).

Slightly prep the incisal margin to a 45° angle (Fig. 44).

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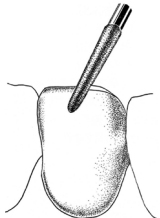
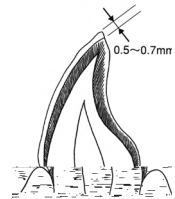
11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan

CE 0123

EC REP **SHOFU DENTAL GmbH**
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.

10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684

www.shofu.com

03:2019-04
71413

ilic
SINCE 1949
QUALITY & VALUE

IVAN ILIC' srl
Via L. Muratori, 46/11 - 20135 Milano (Italy)
TEL: +39 02 36512990 | FAX: +39 02 55016646
MAIL: ilic@ilicdental.com

 ilicdental.com