### Gingitrax-Rex (IT)

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### Gingitrax-Rex (EN)

*How to retract the marginal gingiva without damaging it and protecting it during dental preparation*

*Notes by Prof. Loris Prosper*

#### Main Use

The GingiTrax-Rex retractor is an indispensable tool in various dental procedures:

##### • Fixed Prosthodontics:

- During subgingival preparation with diamond burs.
- For polishing preparations using rubber points.

##### • Conservative Dentistry:

- For working in cervical areas, such as Class V fillings.
- To retract the “free marginal gingiva” and check the adequacy of crown margins.

GingiTrax-Rex Retractors, available sizes: Small - Large

#### Usage Instructions

**GTR Small**, ideal for the following areas:

- Lower arch: incisors and premolars.
- Upper arch: lateral incisors and premolars.

**GTR Large**, ideal for:

- Upper and lower central incisors.
- Larger-than-average lateral incisors.
- All canines.
- All molars.

*Thanks to its versatility, it can also be used in buccal areas.*

#### Warnings for Use

##### Retractor Cord:

During the use of GTR, it is essential to use a retractor cord chosen based on the type of sulcus and gingival morphology:

- Festooned and thin gingiva: prefer retractor cords of sizes 000, 00 (hollow, e.g., Easy-Cord).
- Thick gingival tissue: use retractor cords of sizes 00, 0, 1.

##### Preparation and Insertion:

- To facilitate insertion into the sulci, it is recommended to use 12% chlorhexidine.
- In case of slight bleeding (hemorrhage), impregnate the retractor cord with 25% aluminum chloride (e.g., Viscostat Clear). After stopping the bleeding, rinse away the residue with a gentle stream of water and air.

PERIODENT

sole exclusive importer and Distributor

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